

Spectra S9 plus

Instructions for Use

Please read all instructions carefully before using this product.

Single or double use

Do not discard this manual, please save it for future reference.

## Table of Contents

1. Important Security Measures
2. Important Symbols
3. Indications of Use
4. Components and Assembly
5. Connecting the tubes to the extractor
6. Extraction Guide
7. How to Get Started
8. LCD display and Control Buttons
9. Hygiene and Cleaning
10. How long can I store breast milk?
11. How to defrost and use stored breast milk?
12. Battery information
13. Solving Problems
14. Environment of Use and Storage
15. Technical Specifications
16. Warranty
17. Electromagnetic Compatibility (CEM)

## 1. Important Security Measures

This extractor is designed to be used by a single person, The use of this product by any other person additional to the original purchaser, represents a health risk, affects its performance and voids the product warranty.

(1) If you are a mother infected with Hepatitis B, Hepatitis C or Human Immunodeficiency Virus (HIV), removing your milk does not reduce or eliminate the risk of transmitting the virus to your baby through breast milk.

(2) Before each use, carefully inspect the individual components for cracks, dents, discoloration or deterioration. If damage to individual components is observed, stop using them until the parts have been replaced.



### **READ THIS COMPLETE MANUAL BEFORE USING YOUR MILK EXTRACTOR AND SAVE IT FOR FUTURE REFERENCE.**

When using electrical products, basic safety precautions should always be followed.



### **TO REDUCE THE RISK OF ELECTROCUTION:**

1. Do not use while bathing.
2. Do not place or store the product where it may fall or be thrown away.
3. Do not place in water or where it may get wet with water or other liquids
4. Do not attempt to grasp a product that has fallen into water or other liquids; Unplug it immediately before grabbing it.
5. Do not sterilize the system in microwave oven or immerse the motor in water.
6. If the outer housing of the power adapter loosens or is separating, contact Spectra Baby USA immediately and stop using the product - separation of the housing may result in electric shock.



### **TO REDUCE THE RISK OF BURNS, ELECTROCUTION, FIRE OR INJURIES TO PEOPLE.**














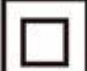
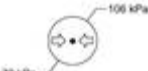



1. This product should NOT be left unattended when plugged into an outlet
2. Do not leave this product unattended near infants or children, Close supervision is necessary when used near them.
3. Use this product only for the use described in this manual. Do not use accessories not recommended by Uzinmedicare Co. /Spectra Baby USA.
4. Never use this product if you have a damaged cable or plug. If it does not work properly, if it has fallen or damaged, it fell into the water, or if any part is broken or lost.
5. Keep the cable away from hot surfaces.
6. Never use it while driving, sleeping or if you are drowsy.
7. Do not use outdoors or operate where aerosols are being used or where oxygen is being administered.

8. Do not disassemble or modify the product.
9. Do not use this product in proximity or on top of other electrical equipment.
10. Allergic reactions may occur.
11. Lint, dust, light (including sunlight), etc. may adversely affect the safety and performance of this product.
12. Always unplug the extractor when not in use.

**Note:**

If you feel chest or nipple pain, turn off the extractor, remove the adapter plug from the power outlet, and break the suction before removing the funnel from your chest, inserting a finger between your chest and the extraction funnel. Put yourself in touch with a health/lactation professional if necessary.

## **2. Important Symbols**

	Sign of general action		Caution/Warning
	Type BF		DC
	On/Off button		Temperature of Use
	Transportation/Storage /Humidity		Keep dry
	Fragile		This product must not be mixed with other commercial waste for disposal.
	Date of Manufacture		Quality Management System
	General Sign of Prohibition		Class II.
	Patmospheric injury		Manufacturer
	Serial Number		Minimum stacking limit

### 3. Indications of Use


The Spectra S9 plus extractor is a breast milk extractor intended for nursing mothers to extract and store their own milk.

### 4. Components and Assembly

Pic

Note: Do not boil for more than 5 minutes as the extraction funnel may deform.

### HOW TO ASSEMBLE THE COUNTERFLOW PROTECTOR

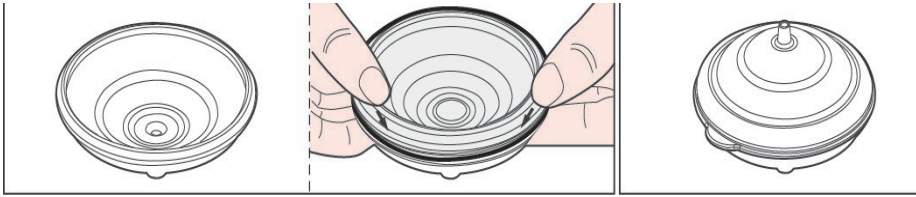

- 1)** Insert the white silicone membrane into the lower base
- 2)** The edge of the silicone membrane should remain on the ring of the lower base to ensure good suction
- 3)** Place the top of the base on the bottom base, ensuring that the 3 pieces fit well to form a perfect seal.

## **5. Connecting the tube system to the extractor (Pic)**

## **6. Extraction guide**

1. Relax, clear your mind, listen to relaxing music. If your baby is not present, you can watch videos or photos of your baby.
2. Stimulate the suction reflex or milk drop before starting extraction. Some women have found that gently massaging their breasts or applying warm compresses from the chest wall to the nipples helps trigger the ejection reflex.
3. Remove your milk from each breast for 10-15 minutes

**NOTE :** For optimal milk extraction, we recommend short and frequent extraction sessions , (10-15 minutes each breast every 2-3 hours, 7-8 times a day)

## **7. How to get started**

Pics

**NOTE:** *Spectra S9Plus is designated to turn off automatically after 30 minutes and stores your current settings for use in your next session. If you want to start with a different configuration than previously used, adjust the new configuration before starting the extraction.*

## **SINGLE OR DOUBLE USE**

***We recommend that you breastfeed your baby whenever he or she can, even when using the milk extractor.***

Massage and Extraction Mode: Press the 3-wave button once to switch from massage mode to extraction mode and vice versa.

### Fashion Massage

To stimulate the "down" reflection of the milk, your baby will suck quickly and superficially, the massage mode is designed to ingest this phase, from the breastfeeding cycle, please adjust the suction level in the massage mode that better stimulates the milk output. Every woman is different, so experiment to find which speed works best and most comfortably for you.

### Extraction Mode

The suction level is measured in mmHg in the machine and refers to the intensity with which it sucks. Be careful when adjusting the suction level of your machine. Some women find that milk extraction is faster at higher levels of suction. However, it is important not to set a suction level so high that it makes you uncomfortable, as pain can affect milk flow.

## 8. LCD display and control buttons

Pic

## 9. CLEANING AND HYGIENIZATION

*Note: Always wash your hands thoroughly with soap and water, rinse thoroughly before handling your extractor and parts.*

- 1) Use for the first time:
  - Sterilize funnels, valves, bottles, caps, nipples. And discs in a large pot of boiling water.
  - Rinse everything thoroughly and allow to air dry.
- 2) After each use:
  - Disassemble all parts or parts that come into contact with your breasts or milk (extraction funnel, backflow protector, valves) and each piece with warm soapy water
    - Rinse everything with clean water and let it air dry.
    - When completely dry, cover until next use.

*Note: Observe the tubes, if you find condensation in the air tubes, wash and dry immediately, or replace them. If you find milk in the tubes, please contact customer service at 855-466-6622 or write to [customerservice@spectrababyusa.com](mailto:customerservice@spectrababyusa.com).*

*To clean the engine, we recommend gently cleaning with a damp (not wet) towel without soap.*

**10. How long can I store breast milk?**

Room temperature:

Temperature	Storage time
15C(60 degrees Fahrenheit)	24 hours
26degrees Celsius (79 degrees Fahrenheit)	4-6 hours
30degrees Celsius (86 degrees Fahrenheit)	4 hours

In the refrigerator:

Temperature	Storage time
0 C (32-39 degrees Fahrenheit)	Up to 8 days

In the freezer:

Freezer Type	Description	Storage time
1 single door	Part of the refrigerator	Up to 2 weeks
2 doors	Separated from the refrigerator, above, below or on one side.	3-4 months
Deep (opens a few times)	Maintains a constant temperature of -17oC(0oF)	6-12 months

<https://espanol.womenshealth.gov/breastfeeding/pumping-and-storing-breastmilk#6>

**11. How to thaw and use stored breast milk**

- Be sure to check the date of the contain before use, Do not use undated milk, or have been longer than recommended.
- Defrost the frozen milk container or bag by putting it in the refrigerator or under the cold water jet.
- Heat the milk under the jet of warm water, gently whisking the bag to mix the milk.
- If warm water is not available, you can heat a pot of water in the stove, Do not boil, once warm, remove the pot from the heat and place the container with the milk in the pot. It can take approximately 20 minutes to bring frozen milk at room temperature, never heat the milk directly on the stove.
- Once the breast milk thawed at room temperature or warmed up after being in the refrigerator or freezer, use it within 2 hours. If you have leftover milk when your baby has finished eating, be sure to discard it 2 hour after you have finished feeding the baby.
- Don't freeze the milk again after you've thawed it. Unused thawed milk can be used within 24 hours, keeping it in the fridge after 24 hours of thawing should be discarded.

## 12. Battery information

The battery used in this product may be at risk of fire or electrical burns if not used properly.

Do not disassemble, heat, incinerate or dispose of your battery on fire.

Dispose of this battery in accordance with local, state or federal regulations.


<b>RECHARGEABLE BATTERY 7.4V - 900mAh LITIO-POLIMERO</b>	
<b>Battery indicator</b>	<b>Time to use/extract</b>
3 bars	2 hours
1 bar	30 Minutes
Flashing light	Less than 10 minutes; you must recharge the battery.

## 13. STIRRING PROBLEMS




Problem	Possible Causes	Check	Solution
No electric current	Power Cable or Adapter	Check if the power cord is not connected or damaged.	Make sure the cable is connected to the machine. Connect it to the power outlet. If it is damaged, replace it. Or contact your seller
	Dead battery	Is the battery fully charged?	If it is not charged contact your seller
Doesn't suck	Air pipes	Check that the tube is connected to the machine. Check if the air tube is damaged.	Connect the air tube to the machine. If the air tube is damaged, replace it.
	Valve	Check that the valve is properly connected to the extraction funnel.	Attach the valve to the funnel: Turn the valve to secure it.  check if the backflow guard is damaged.  Check for membrane damage.
	Air outlet	Is the air outlet closed?	Close the air outlet.
Low suction	Suction level	Check the suction level	Increase the suction level.
	Defective assembly	Is the system properly assembled?	Re-assemble the pieces again
	Suction Funnel	Check your extraction funnel, if it is damaged or is the right size	Confirm the measurement of your funnel using the measurement guide in <a href="http://spectrababyusa.com">spectrababyusa.com</a> ;  If it is damaged, replace it.
	Valve	Check the valve for any damage or crack. Check the membrane, and replace if it is damaged.	Replace the valve.
Pain	Suction level	Very high suction	Adjusting at lower speed
	Suction funnel	Check your extraction funnel, if it is damaged or is the right size	Confirm the measurement of your funnel using the measurement guide in <a href="http://spectrababyusa.com">spectrababyusa.com</a> ;  If it is damaged, replace it.
	Cycle speed	Check the speed	Change the speed to a comfortable level.

#### 14. Environment to Use and Store

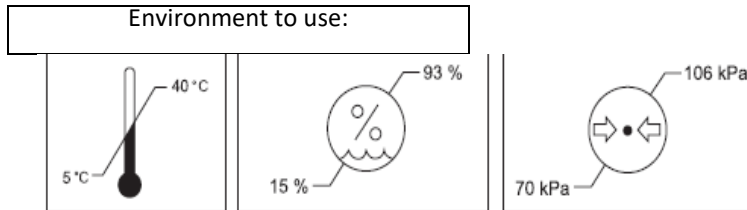

Temperature Humidity Pressure

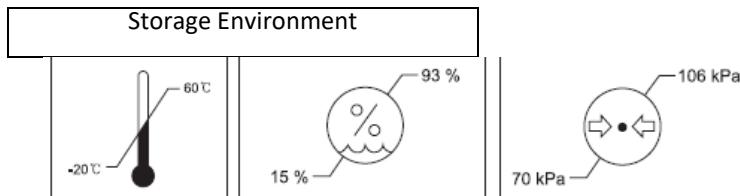

Temperature Humidity Pressure

## 15. TECHNICAL SPECIFICATIONS

Protection Type: Class II.

Weight: 280 grams

Consumption current: 24VA

Input voltage: 100-240 V- 50/60 Hz, 500mA

Output voltage: 7.4V 900AhLithium-polymer

1. Type of electric shock protection: Class II equipment
2. Degree of protection against electric shock: Applied parts type BF
3. Classification according to the degree of protection against water inlet: IP22
4. This equipment is not suitable for use in the presence of flammable anesthetics or oxygen
5. Operating mode: continuous operation.

## 16. Warranty

This product is guaranteed by Spectra baby usa, so that the buyer is free from charges for defects in material and workmanship for a maximum period of two (2) years from the date of purchase. In the event of a defect or breach of this warranty, Spectra baby uses, will replace this product free of charge for such replacement or parts at the Spectra baby office used listed below. the buyer will assume full responsibility and return costs for this product, including the risk of loss prior to receipt by Spectra baby uses, as well as shipping, packaging and insurance costs.

Each and every warranty involved is limited to a duration of two(2)years from the date of purchase. some states do not allow limitationson how long a warranty lasts, so the above limitations may not apply to you.

Spectra baby's liability is used, for breach of any written or implied warranty, limited to the replacement of this product. Spectra baby uses no liability under any circumstances for any incidental or consequential damages. some states do not allow the exclusion or limitation of incidental or consequential damages, so the above limitations may not apply to you.

This warranty gives you specific legal rights. you may have other rights that vary from state to state.

SPECTRA BABY USA shall have no obligation under this warranty to replace this product and/or its components due to any malfunction or damage caused by accident or arising from misuse, abuse, improper maintenance, unauthorized modification or connection to an incorrect power source. Without limiting the generality of the foregoing, the fall of this product or its components and the visible cracking of the equipment housing shall be presumed as defects resulting from misuse or abuse. Accessories not expressly manufactured by SPECTRA BABY USA and intended for use with the Spectra S3 Pro milk extractor will immediately void this warranty. A replacement fee for damaged parts will be charged.

If you wish to make a claim under this warranty, please contact the warranty staff at [warranty@spectrababyusa.com](mailto:warranty@spectrababyusa.com) for instructions, or visit our website to complete such claim.

You will be asked to provide proof of purchase or receipt.

Spectra Baby USA  
Recycling  
2180 SW 71st Terrace  
Davie, FL 33317  
Website: <http://www.spectrababyusa.com>

Email: [customerservice@spectrababyusa.com](mailto:customerservice@spectrababyusa.com)

Telephone: 855-446-6622

## **17. COMPATIBILIDAD ELECTROMAGNETICA (EMC)**

This equipment needs special precautions regarding its electromagnetic compatibility and must be installed and put into service in accordance with the information provided in the user manual.

Wireless communications equipment, such as wireless home network devices, mobile phones, wireless phones and their base stations, walkie-talkies can affect this equipment and must be maintained at least 1.0m away from theequipment.

This equipment has been verified and meets the requirements of Part 15 of the FCC Rules.

Your operation is subject to these two conditions:

- A. This equipment cannot cause harmful interference.
- B. This equipment must accept any interference received, including interference that may cause malfunction.

This equipment has been verified and has been proven to meet the limits for medical devices in IEC 60601-1-2:2014. These limits are designed to provide reasonable protection against harmful interference in a typical medical facility. This equipment generates, uses and can radiate radio frequency energy and if not installed and used in accordance with the instructions, may cause harmful interference to other nearby devices. However, there is no guarantee that interference will not occur at a particular facility. If this equipment causes harmful interference to other devices, which can be determined by turning the equipment off and on, the user is advised to attempt to correct the interference by one or more of the following measures.

- Reorient or relocate the receiving device.
- Increase separation between teams.
- Connect the equipment to a power outlet on a circuit different from that to which the other devices are connected. Consult your manufacturer or service technician for assistance.

**Distributed in USA by:**

Spectra Baby USA

Recycling

2180 SW 71st Terrace

Davie, FL 33317

Website: <http://www.spectrababyusa.com>

Email: [customerservice@spectrababyusa.com](mailto:customerservice@spectrababyusa.com)

Telephone: 855-446-6622